Blue Phoenix® Multifunctional Cabinet Vaccine Sprayer

AUTOMIST ALL DIGITAL AUTOMATED SPRAY VACCINATOR

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Blue Phoenix automated spray vaccinator is a very adaptable and convenient method of administering live vaccines directly to day old poultry chicks simultaneously by standard Multi-Routes method of vaccine administrating.

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Next generation of All-Electric and electronic vaccinator instruments.

No pneumatic, no Air Consumption, driven.

Medical grade material and design → no biofilm build up & no cross contamination of vaccine mix

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Routes: Intra-tracheal, eye spray, intra-nasal, Intra-pulmonary, Intra-Air sac.

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Fully-enclosed chamber minimizes vaccine waste and safely protects operators.

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Target:

For spray vaccination of one day old chicks (optional gel or water based vaccines).

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Anaerobic electrostatic atomizer technology:

This feature is optional and can be activated on digital controller.

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Nozzle:

4 pcs. For one formulated vaccine administration in each spray.

6 pcs. For two formulated vaccine simultaneously administration in each spray.

4 Nozzles for maximum coverage of all birds, four spray nozzles ensure that every chick is evenly and completely coated with vaccine.

Nozzle Adjustment to match crate size.

Specially designed nozzle for Vaccination of one day old chicks with 140 (110 – 190) μm in diameter droplets (at static pressure of 12 bar).

Spatial multidirectional rotation of droplet technology.

Flow rate sum of 4 nozzles at pressure of 12 bar: 360 ml/min

In settable pressure version, droplet size can be set in a range of 5 to 300 μm .

It spray a wide range of settable droplet size in a uniform aerosol, dry mist, wet mist and fog.

Fast shut-off, AntiDrip, designed nozzle, prevents wasting of vaccine and contamination of hoses (no cross contamination).

Easy detachable nozzles for easy access for better cleaning.

Nozzle Replacements and mounting are exclusively designed so that can be easily replaced by any non-technical personal thereby keeping maintenance as low expense as possible.

100-mesh vaccine filtration minimizes contamination and maximizes chick survivability (optional on customers order, it reduces spray velocity and flow rate, and may change droplet size, and should not be used on coccidiosis and gel based vaccines).

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Nozzle position Adjustment

* For adjustment of nozzles based on your chick box size and local standards please contact your supplier.

Smooth crate handling on stainless steel guides.

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Hoses:

Neutral hoses which don't destroy vaccine particles.

High quality, high pressure resistance and anti-Biofilm hoses.

Minimum reservoir in transfer path, healthier chicks: Easier and faster cleaning, with reduced potential for cross-contamination.

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Dosing:

Simple and easy adjustable spray time (Minimum time: 0.01 sec.), dosage and droplet size (this feature is fitted just in ultra-version) just by touch on Smart digital controller, based on chick sizes and vaccine types with minimum downtime.

Accurate and easy to set, unlimited settable dose.

Dosage Chart can be received from technical team.

A Clear and common dosage setting chart is provided on the machine which helps in quick dosage setting.

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Note. : Dose and pressure needs to be re-adjusting before when medication needs to be changed.

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Capacity:

Inoculate up to 60,000 chicks per hour (approx.) in regular use with an associated conveyor but depends on set dose and operators chick box handling speed.

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Mode of Operations:

Standard Multifunctional method of vaccine administrating.

The Spray can be controlled by unlimited modes of operation.

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Trigger:

Multi-Mode Operations: The Spray can be controlled by 3 modes of operation.

Auto: In which the tray trigger, fires the nozzles (Each time, after chick box placement in cabin of spray, limit switch trigger fires)

Pedal: In which foot trigger, fires the nozzles

Manual: Black Push Button, fires the nozzles (Manual button is used for priming / flushing)

Soft Trigger.

The trigger is encapsulated in an IP 68 casing.

www.dana-enterprises.com

Also the sensitivity of the switch decreases operator fatigue and increases operation velocity.

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Medicine Cooling Container, Provided with cooling gel to maintain temperature.

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Motor:

High tech transfusion-diaphragm pump (accurate dose and pressure control).

Pressure: 12 – 15 Bar.

In settable pressure version, pressure can be set in a range of 5 to 25 Bar.

Transfusion Dosing Pump to deliver the vaccine and medication in accurate amounts.

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Powered by Backlit LED Display for Crystal Clear viewing in dark chick rooms (optional).

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Stainless steel collection plate installed under spray cabin to prevent floor getting wet and dirty.

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Dimensions: (Lx*Wx*Hx) - 103 × 62 cm × 42 cm

Hood dimensions: (Lx*Wx*Hx) - 71 x 54 x 35 cm

Sturdy stainless steel frame OR anti-scratch and antirust blast furnace color.

Food-grade, corrosion-resistant construction materials help to maximize chick health.

Long Lasting user friendly equipment.

Electrical system ensures continues linear (without oscillation) operation.

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Spare parts:

Consumable spares kit available. Tool Kit available.

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Due to continuous product development specifications are subject to change and availability of different versions of our product you should inform us in every contact about your own device.

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Phoenix Industries[®]. Based in Silicon Valley, San Francisco, California, USA

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California Made / Made in the U.S.A